

INSTRUCTION MANUAL FOR DIFFERENTIAL BLOOD CELL COUNTER DBC-9

1. By counter-clock wise way turns both of the knobs. It makes the digits to "0".
2. There are three digits in each window and eight windows are totally in the unit. The right side of the window is an aggregated count window and it can not be numbered by any key. Other windows: BASO., EOSIN., MYELO., JUVEN., STAB., SEG., LYMPH., and MONO. are all countable individually by key.
3. At the upper of each window has shown the programs of WBC and GRAPHS respectively. It enables the inspector to count in accordance with the line up position or on his/her own habitual adjustment and line up as well.
4. When the WBC in each window comes to number "100", a sound of "DON" to be heard. This manner extends to number "900" (such as "100", "200", "300" "900").
5. Each window shows the digits that is a differential value for WBC.

Note:

1. The user must press each key full down to the bottom. Otherwise, an error numbers would be found.
2. If total summary is not correct or when you press a key, the number doesn't increase, please make sure if you press each key full down to the bottom. If you do press each key full down the bottom, and the problems are still there, please contact your distributor for repair.

Maintenance and care:

Keep the counter clean all the time. Cleaning can be done with a damp cloth. Avoid the use of solvents as they may damage the product.